



ClearSounds®

Hear how good life can be



## Operating Manual

**ClearSounds® A600 DECT 6.0  
Amplified Freedom™ Phone  
with Full ClearDigital™ Power**

ENGLISH

ESPAÑOL

FRANÇAIS

# TABLE OF CONTENTS

## PRIOR TO USE

- Pg. 4 ..... Congratulations
- Pg. 5-6 ..... Important Safety Instructions
- Pg. 7-9 ..... Location of Controls
- Pg. 10-11 .... Keys Descriptions
- Pg. 12 ..... LCD Icons Descriptions
- Pg. 13-16 .... Installation Steps

## MENU NAVIGATION

- Pg. 18-19 .... Menu Map

## BASIC PHONE FUNCTIONS

- Pg. 19-20 .... Handset Registration
- Pg. 20-21 .... Amplification
- Pg. 22 ..... Speakerphone
- Pg. 22-23 .... Ringer Setting
- Pg. 24 ..... Handset Power On/Off
- Pg. 24 ..... Hearing Aid and Cochlear Implant Users
- Pg. 25 ..... Answer a Call
- Pg. 25-26 .... Make a Call

## PHONE BASE SETTINGS

- Pg. 27 ..... Dialing Mode
- Pg. 27-28 .... Call Waiting/Flash
- Pg. 28 ..... Modify PIN
- Pg. 29 ..... Delete/Terminate Handset
- Pg. 30 ..... Base Reset

## PHONE HANDSET SETTINGS

- Pg. 30-31 .... Name Handset/ Reset Handset
- Pg. 31 ..... Select Base
- Pg. 31-32 .... Out of Range Alarm
- Pg. 32 ..... Auto Answer
- Pg. 32-33 .... Audio Set-up
- Pg. 33-34 .... Talking Set-up

- Pg. 34 ..... Keys Tones Set-up
- Pg. 35 ..... One-Touch Memory
- Pg. 36-37 .... Mute, Redial
- Pg. 37-38 .... Languages

## DISPLAY SETTINGS

- Pg. 38-39 .... Set Contrast
- Pg. 39-40 .... Set Date & Time
- Pg. 40 ..... LCD Description
- Pg. 40 ..... Reviewing Voice Mail Messages

## CALLER ID (CID)

- Pg. 41 ..... General Information
- Pg. 41 ..... Receiving & Storing Caller ID Records
- Pg. 42 ..... Talking Caller ID
- Pg. 42 ..... Caller ID on Call Waiting
- Pg. 43 ..... Review Caller ID
- Pg. 43 ..... Dial from Caller ID
- Pg. 44 ..... Delete Caller ID
- Pg. 44-45 .... Store Caller ID

## PHONE ANSWERING MACHINE (TAM)

- Pg. 45-46 .... General Information/Keys Descriptions
- Pg. 46 ..... Turn On/Off
- Pg. 47-49 .... TAM Settings
- Pg. 49-52 .... Announcement Options
- Pg. 52-54 .... Message Options
- Pg. 55 ..... Record a Memo
- Pg. 55-57 .... Remote Code & Remote Access
- Pg. 57 ..... TAM Reset

## PHONEBOOK

- Pg. 57-58 .... General Information
- Pg. 58-59 .... Add Phonebook Entries
- Pg. 59 ..... Using Pause
- Pg. 59 ..... View Phonebook Entries
- Pg. 60 ..... Dial a Phonebook Number
- Pg. 60-61 .... Phonebook Entry, Edit, Delete, Status

## ADDITIONAL HANDSET FUNCTIONS

Pg. 62-63 .... Intercom/Conference Call

Pg. 63 ..... Paging

Pg. 63 ..... Key Lock

Pg. 64 ..... Alarm

Pg. 64 ..... Torch

## TROUBLESHOOTING GUIDE

## TECHNICAL SPECIFICATIONS

## REGULATORY COMPLIANCE

Important FCC Information for Customers

## INDUSTRY CANADA

## WARRANTY POLICY

### **PRIOR TO USE**

Congratulations on your purchase of the A600 phone from ClearSounds Communications®. Please read this User Manual carefully in order to get the best use out of your new phone. This phone is designed for individuals with mild to severe hearing loss. Keep this manual near the phone for easy reference.

Please visit our website at [www.clearsounds.com](http://www.clearsounds.com) to register your product, download additional manuals and review troubleshooting tips.

Be sure to save your sales receipt as your proof of purchase should you need warranty service. The serial number for this product is located on the underside of the base and looks like: A600\_SIL\_XXX000XXX00.

**Warranty Service** – This product is designed to provide years of quality service. However, if there is a malfunction and the troubleshooting recommendations on pages 65-66 do not resolve the problem, follow the warranty procedure on pages 70-71.

## ■ IMPORTANT SAFETY INSTRUCTIONS

Read and understand the Installation Guide and User Manual. When using your phone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to person, including the following:

**WARNING:** The A600 phone is capable of amplifying sounds to a loud volume. It is important to instruct all potential users of its proper operation. It is advised to adjust the volume control to the minimum level (Low) when not in use and alert other users that hearing damage can potentially result from misuse.

**WARNING:** Use only the power supply indicated in the User's Manual.

**WARNING:** To avoid the possibility of electrical shock, do not use this product near water (For example, near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement or near a swimming pool) while in the bathtub or shower or when you are wet. If the phone becomes submerged in water, do not attempt to retrieve it until after you have unplugged the AC adapter from the electrical outlet and the line cord from the modular wall jack. Do not plug the phone back in until it has dried completely.

**WARNING:** Avoid using the phone during electrical storms. There may be a remote risk of electric shock from lightning. If the electrical storm is present in your immediate area, possibility of electrical shock exists.

**WARNING:** Do not use the phone if you suspect a gas leak in the area. The phone's electrical contact could create a spark, which could ignite any heavy concentration of leaking gas.

**WARNING:** When cleaning, unplug the phone first, and then use a damp cloth to wipe. Do not use liquid cleaners or aerosol cleaners on the phone.

**WARNING:** No phone should ever be placed near an open radiator or heating register.

**WARNING:** Occasionally a power surge can occur on electrical lines without warning. With any electrical device that you plug

into an outlet, it is suggested you use a surge protector. A surge protector will prevent power surges from damaging your phone. Surge protectors are simple devices available at your local hardware store.

**WARNING:** Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.

**WARNING:** Never push any object through the slots in the phone. They can touch dangerous voltage points or short out parts that could result in risk of fire or electrical shock.

**WARNING:** Do not cover slots and openings on the units, they are needed for ventilation and protection against overheating.

**WARNING:** Stop using the phone if dropped or if it becomes damaged.

**WARNING:** If you or a member of your household has a pacemaker or other medical devices, consult your physician or the manufacturer of the pacemaker or medical device before using this product. The product should be used in accordance with the guidelines provided by your physician or the manufacturer of the pacemaker or medical device.

**WARNING:** Ring volume is very loud and can damage your hearing. Never put your ear close to the ringer.

**WARNING:** Pay attention to the polarity of the batteries and insert the rechargeable batteries in accordance with the polarity symbols. Use only batteries identified as acceptable and never use alkaline batteries, which can cause personal injury as well as damage to the unit.

### **Hearing Aid Compatibility (HAC)**

This phone meets the FCC standards for Hearing Aid Compatibility.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR WARRANTY AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.



































































































































































































































































































































































































































