

KEY DESCRIPTIONS

Handset



Dial Key

Make/Answer a call. Activates the speakerphone if pressed during a call.







End Key

End a call. Long press to activate power on/off.



Navigation Key:

-  Up scroll for lists and menu options
Increase the earpiece/speakerphone volume during a call.
Enter the Caller ID (CID) book.
-  Down scroll for lists and menu options
Decrease the earpiece/speakerphone volume during a call.
Enter the Phonebook.
-  Set up an internal call to another handset
Transfer an incoming call to another handset.
Set up a conference call with an external line and an internal handse.
-  Enter the redial list.
Insert a pause when pre-dialing a number or storing a phonebook entry.



Soft Keys – Two soft keys are used to select functions displayed directly above each of the soft keys (on the bottom line of the display). Functions displayed above the soft key will change as you move through the menu.



Star Key - Press and hold to turn on/off keypad lock function.
Set up a conference call.



One-touch memories – Store up to 4 numbers for one-touch memory dialing.



O Key – In standby mode, press and hold this key until the icon appears to activate the shaker and visual ringer function. Press and hold this key again until the icon disappears to deactivate it.

MENU MAP (p. #) denotes page in Operator Manual

Main Menu	Submenu I	Submenu II
CID (Caller ID) Book (p. 36)	Add to PB(Phonebook) (p.39) Delete (p.39) Delete All (p.39)	
Phonebook (p. 49)	New (p.50) Edit (p.52) Delete (p.52) Delete All (p.52) PB Status (p.53)	
BS (Base) Settings (p. 23)	Terminate HS (Headset) (p.25) Dial Mode (p.23) Flash Time (p.23) Modify PIN (Personal Identification Number) (p.24) BS Reset (p.25)	
HS (Handset) Settings (p. 26)	Alarm (p.55)	
	Audio Setup (p.29)	Speaker V.(volume) (p.29) Earpiece V.(volume) (p.29)
	Ring Setup (p.19)	INT (Internal) Ring (p.19) EXT (External) Ring (p.19)
	Tone Setup (p.28,30)	Key Tone (p.30) Range Alarm (p.28)
	Language (p.33) HS Name (p.26) Auto Answer (p.28) LCD Contrast (p.34)	
	Date & Time (p.34-35)	Date Format (p.34) Time Format (p.34) Set Date (p.35) Set Time (p.35)
	Select Base (p.27) HS Reset (p.27)	
Registration (p. 17)		

TELEPHONE ANSWERING MACHINE MENU MAP

Main Menu	Submenu I	Submenu II
Announce (p.42)		
Message (p.45)	New MSG (p.45) Saved MSG (p.45) Delete All (p.46) Memo Record (p.46)	
TAM Setting (p.40)	TAM On/Off (p.40) TAM Alert (p.42) ANNC Select (p.44) Set Ring (p.40) TAM Language (p.41) Remote Code (p.47) TAM Default (p.49)	



ClearSounds®
Hear how good life can be



QUICK START GUIDE

ClearSounds® A300

DECT 6.0 Cordless Phone


with Sound Boost

Service in United States and Canada, contact:
ClearSounds Communications
Attention: Repair Center
1743 Quincy Avenue, Suite 155
Naperville, IL 60540 USA

800-965-9043 or www.clearsounds.com
Printed manual available upon request

PARTS CHECKLIST

PHONE BASE

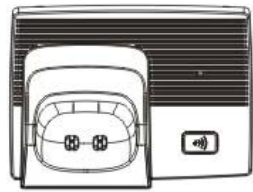

CORDLESS HANDSET



BATTERIES



PHONE LINE CORDS



CARRYING CLIP



USER'S MANUAL








AC ADAPTER



REGISTERING ADDITIONAL HANDSETS

The supplied handset is already registered to the base unit, and you can register up to 4 additional handsets to the base.

1. Before registering an additional handset to the base, you should press and hold the page key , which is located on the main base, for about 5 seconds and then release it.
2. Press the **Menu** soft key on the handset you are registering to display the main menu list.
3. Press up  or down  to scroll to **Registration** and then press the **Select** soft key.
4. Press up  or down  to scroll through the base list and select one of the bases and then press the **Select** soft key. The handset will search for the requested base.
5. If the handset is successfully registered, you will be prompted to enter the PIN code (the default PIN is 0000). Upon validation of the PIN, the Registration tone will sound and the base assigns a number to the handset.

More information on this feature located in the Operating Manual on page 17.


CONNECTING YOUR PHONE

1. CHECK PARTS LIST

2. PHONE INSTALLATION: The phone should be located in an area where it is easy to see the visual display and lighted ringer. The phone may be placed on a shelf or table or may be fixed to the wall. **Begin by removing the clear plastic Caller ID protective tab. This film tab is only for display and to protect your Caller ID display during shipping.**

Desk / Table Mounting

1. Connect one end of the 6 ft phone cord to the line jack on the back of the base and connect the other end to a modular phone jack.
2. Plug the AC power adapter into the jack on the bottom of the base, then plug the AC adapter into a standard electrical outlet or surge protector.

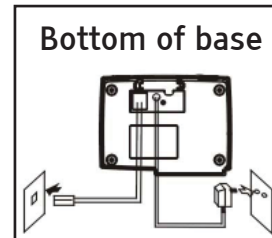

NOTE: The adapter is required for the phone to work.

Wall Mounting

1. Install two self-tapping screws (not supplied) at a distance of 86.5 mm apart from each other on a horizontal line.
2. Plug the AC power adapter into the jack on the bottom of the base. **NOTE:** The adapter is required for the phone to work.
3. Connect one end of the 8-inch phone cord to the line jack on the bottom of the base and connect the other end to a modular phone jack.
4. Holding the base slightly above the mounting screws on the wall jack, push the base against the mounting screws so they are hooked into the left and right keyhole slots on the back of the phone. Slowly slide the base down until it snaps into place.
5. Plug the AC power adapter into the electrical outlet. **CAUTION:** To reduce risk of personal injury, fire, or damage use only the adapter provided.

If at any time you have questions about your telephone after reading this manual, please contact our ClearSounds® Customer Service Department at 1-800-965-9043.

LOCATION OF CONTROLS

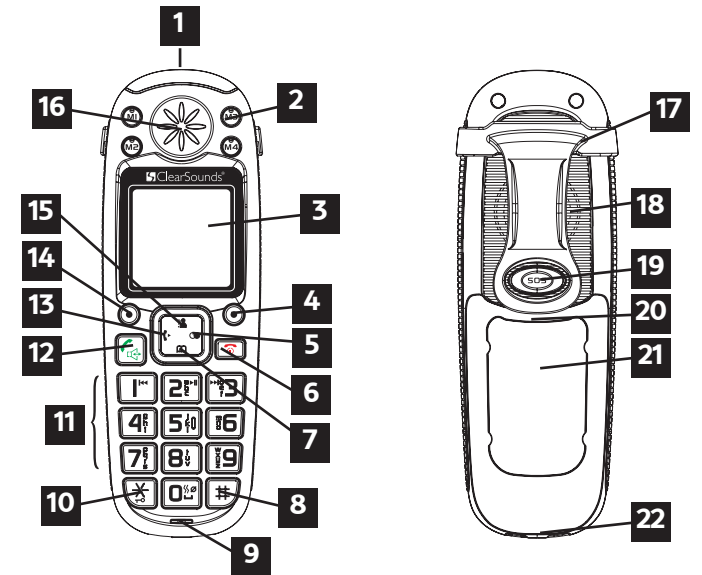


Figure 1

- | | |
|--------------------------------|------------------------------|
| 1 Visual Phone Ringer | 12 Dial/Speakerphone Key |
| 2 (4) 1-Touch Memory Dial Keys | 13 Left/Intercom Key |
| 3 Large LCD/Caller ID Screen | 14 Left Soft Key |
| 4 Right Soft Key | 15 Up/Caller ID Key |
| 5 Right/Redial Key | 16 Earpiece |
| 6 End Key | 17 Carrying Clip |
| 7 Down/Phonebook Key | 18 Speaker |
| 8 Hash (Pound) Key | 19 SOS Key |
| 9 Microphone | 20 Screw Eye |
| 10 Star Key | 21 Battery Compartment Cover |
| 11 Lighted Keypad | 22 Charging Contacts |




MEMORY KEYS

Dialing from the Memory M1 – M4 and SOS keys

Press one of the **M1 – M4** or **SOS** memory dial keys and the phone will automatically begin to dial (Page 30 of the Operating Manual describes how to program the memory dial keys).

TELEPHONE ANSWERING MACHINE (TAM) OPERATION

To Turn the Answering Machine ON/OFF

1. Press the **TAM** soft key to display the TAM main menu list.
2. Press down  to scroll to **TAM Setting** and then press the **Select** soft key.
3. Press the **Select** soft key to select **TAM On/Off**.
4. Press up  or down  to select **Off** or **On** and then press the **Select** soft key.