

KEY DESCRIPTIONS

Handset



Dial Key

Make/Answer a call

Activates the speakerphone if pressed during a call



End Key

End a call

Long press to activate power on/off



Navigation Key:



Up scroll for lists and menu options

Increase the earpiece/speakerphone volume during a call

Enter the Caller ID (CID) book



Down scroll for lists and menu options

Decrease the earpiece/speakerphone volume during a call

Enter the Phonebook



Set up an internal call to another handset

Transfer an incoming call to another handset

Set up a conference call with an external line and an internal handset



Enter the redial list

Insert a pause when pre-dialing a number or storing a phonebook entry



Soft Keys – Two soft keys are used to select functions displayed directly above each of the soft keys (on the bottom line of the display). Functions displayed above the soft key will change as you move through the menu.



Star Key - Press and hold to turn on/off keypad lock function

Set up a conference call



One-touch memories – Store up to 4 numbers for one-touch memory dialing



O Key – Press and hold to activate the shaker and visual ringer function

MENU MAP (p. #) denotes page in Operator Manual

Main Menu	Submenu I	Submenu II
CID (Caller ID) Book (p.41)	Add to PB (p.44) Delete (p.44) Delete All (p.44)	
Phonebook (PB) (p.57)	New (p.58) Edit (p.60) Delete (p.61) Delete All (p.61) PB Status (p.61)	
BS (Base) Settings (p.26)	Terminate HS (p.29) Dial Mode (p.27) Flash Time (p.27) Modify PIN (Personal Identification Number) (p.28) BS Reset (p.29)	
HS (Handset) Settings (p.30)	Audio Setup (p.32) Ring Setup (p.22) Tone Setup (p.34) Language (p.37) HS Name (p.30) Auto Answer (p.32) LCD Contrast (p.38) Date & Time (p.39) Select Base (p.31) HS Reset (p.30) Amplified (p.21) Talk Keypad (p.34) Talk CID (p.33)	Earpiece V. (vol.) (p.32) INT (Internal) Ring (p.23) EXT (External) Ring (p.23) Key Tone (p.34) Range Alarm (p.31) Date Format (p.39) Time Format (p.39) Set Date (p.39) Set Time (p.39)
Registration (p.19)		

TELEPHONE ANSWERING MACHINE MENU MAP

Main Menu	Submenu I	Submenu II
Announce (p.49)		
Message (p.52)	New MSG (p.53) Saved MSG (p.53) Delete All (p.54) Memo Record (p.55)	
TAM Setting (p.47)	TAM On/Off (p.46) TAM Alert (p.48) ANNC Select (p.48) Set Ring (p.47) TAM Language (p.47) TAM Default (p.57)	



ClearSounds®
Hear how good life can be



QUICK START GUIDE


ClearSounds® A600 DECT 6.0
Amplified Freedom™ Phone
with Full ClearDigital™ Power

Service in United States and Canada, contact:
ClearSounds Communications
Attention: Repair Center
1743 Quincy Avenue, Suite 155
Naperville, IL 60540 USA

800-965-9043 or www.clearsounds.com
Printed manual available upon request

PARTS CHECKLIST

PHONE BASE

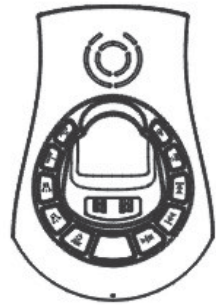

CORDLESS HANDSET



BATTERIES



PHONE LINE CORDS



CARRYING CLIP



BRACKET



AC ADAPTER



USER'S MANUAL



REGISTERING ADDITIONAL HANDSETS

The supplied handset is already registered to the base unit, and you can register up to 4 additional handsets to the base.

1. Before registering an additional handset to the base, you should press and hold the page key (☎), which is located on the main base, for about 5 seconds and then release it.
2. Press the **Menu** soft key on the handset you are registering to display the main menu list.
3. Press up (↑) or down (↓) to scroll to **Registration** and then press the **Select** soft key.
4. Press up (↑) or down (↓) to scroll through the base list and select one of the bases and then press the **Select** soft key. The handset will search for the requested base.
5. If the handset is successfully registered, you will be prompted to enter the PIN code (the default PIN is 0000). Upon validation of the PIN, the Registration tone will sound and the base assigns a number to the handset.

More information on this feature located in the Operating Manual on page 19.

CONNECTING YOUR PHONE

1. CHECK PARTS LIST

2. PHONE INSTALLATION: The phone should be located in an area where it is easy to see the visual display and lighted ringer. The phone may be placed on a shelf or table or may be fixed to the wall using the wall mounting bracket. **Begin by removing the clear plastic Caller ID protective tab. This film tab is only for display and to protect your Caller ID display during shipping.**

Desk / Table Mounting

1. Connect one end of the 6 ft phone cord to the line jack on the back of the base and connect the other end to a modular phone jack.
2. Plug the AC power adapter into the jack on the back of the base, then plug the AC adapter into a standard electrical outlet or surge protector.

NOTE: The adapter is required for the phone to work.

TALKING SETTING

Caller ID

Your ClearSounds® A600 comes from the factory preset for Talking Caller ID Mode set to ON and will announce the caller's number between rings. To change this setting:

1. Press the **Menu** soft key to display the main menu list.
2. Press up (↑) or down (↓) to scroll to **HS Settings** and then press the **Select** soft key.
3. Press up (↑) or down (↓) to scroll to **Talk CID** and then press the **Select** soft key.
4. Press up (↑) or down (↓) to select **On** or **Off** to activate/deactivate the Talking Caller ID function and then press the **Save** soft key.

MEMORY KEYS

Dialing from the Memory M1 – M4 and SOS keys

Press one of the **M1 – M4** or **SOS** memory dial keys and the phone will automatically begin to dial (Page 30 of the Operating Manual describes how to program the memory dial keys).

If at any time you have questions about your telephone after reading this manual, please contact our ClearSounds® Customer Service Department at 1-800-965-9043.

LOCATION OF CONTROLS

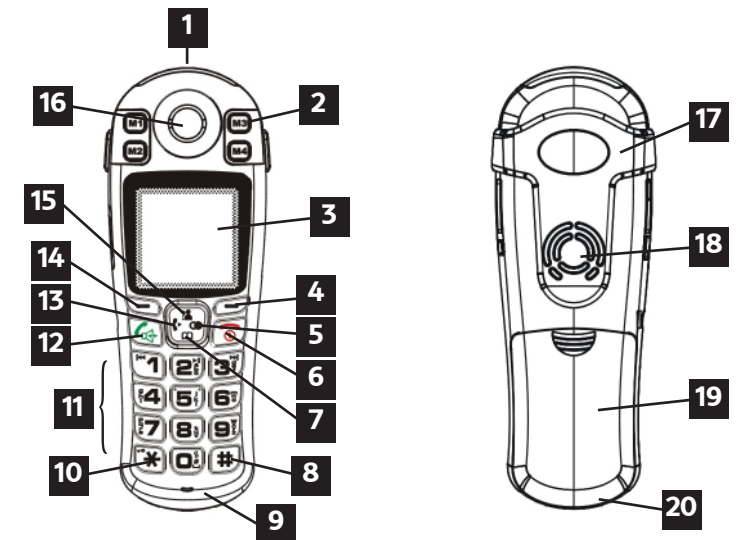


Figure 1


- | | |
|---|------------------------------|
| 1 Visual Phone Ringer & Amplify Indicator Light | 11 Lighted Keypad |
| 2 (4) 1-Touch Memory Dial Keys | 12 Dial/Speakerphone Key |
| 3 Large LCD/Caller ID Screen | 13 Left/Intercom Key |
| 4 Right Soft Key | 14 Left Soft Key |
| 5 Right/Redial Key | 15 Up/Caller ID Key |
| 6 End Key | 16 Earpiece |
| 7 Down/Phonebook Key | 17 Carrying Clip |
| 8 Hash (Pound) Key | 18 Speaker |
| 9 Microphone | 19 Battery Compartment Cover |
| 10 Star Key | 20 Charging Contacts |

TELEPHONE ANSWERING MACHINE (TAM) OPERATION

Method 1: Using the Handset

1. Press the **TAM** soft key to display the TAM main menu list.
2. Press down (↓) to scroll to **TAM Setting** and then press the **Select** soft key.
3. Press the **Select** soft key to select **TAM On/Off**.
4. Press up (↑) or down (↓) to select **Off** or **On** and then press the **Select** soft key.

Method 2: Using the Base

1. With power coming to the base, press the  key on the base to turn the answering machine **Off** or **On**. The base will announce the selected mode.