

i-700


Quick Guide

The Phone Layout

Handset



Base

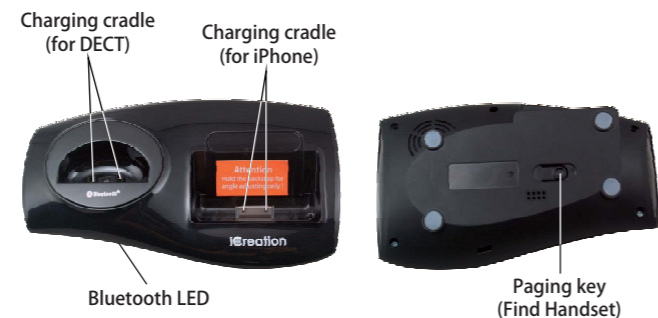

Key Functions

	Press to enter # or to switch between upper case and lower case letters in the English mode.
	Press and hold to turn on or off the keypad lock function.
	Select keys: Press to perform the function listed above the key.
	Press to turn on the phone. If you press and hold, the phone turns off. While navigating through menus, briefly press it to cancel your input and return to the main screen. During a call, briefly press it to end the call.
	Press to place and answer landline calls.
	Press to place and answer iPhone calls.
	Press to activate the speakerphone for handsfree operation during a call.
	Press to scroll up or down when browsing menus. In standby mode: <ul style="list-style-type: none"> Left: Press to place an internal call. Right: Press to review outgoing calls list. Up: Press to view incoming calls list. Down: Press to review contacts. During use: <ul style="list-style-type: none"> Up: Increases earpiece volume. Down: Decreases earpiece volume.

Charging

DECT Phone & iPhone

Before initial operation, fully charge the DECT phone for 15 hours or more.

To charge the DECT phone or iPhone, simply place it on the base.



Notes:

- When charging, the bars in scroll.
- You can adjust the visual angle of the iPhone according to your needs (from 30 degrees to 75 degrees).

Turning On or Off the Phone

When you place the handset (DECT phone) on the base, it automatically turns on.

To turn on the handset (when it's off), press .

To turn off the handset, press until "Goodbye!" appears.

Note:

- Nothing appears on the screen when the battery power is very low. Fully charge the handset before use.


Bluetooth

Pairing with an iPhone

Turn on the Bluetooth feature of your iPhone. For example, for an iPhone4, click **Settings, General, Bluetooth** and **OFF**, then the iPhone searches for available devices.

	Open the menu.
And	Select the Bluetooth menu.
,	The LCD shows Please wait... , and the Bluetooth LED flashes.

When the iPhone finds i-700, click **Not Paired** to connect it with the base. When the iPhone is successfully connected, **Connected** displays.

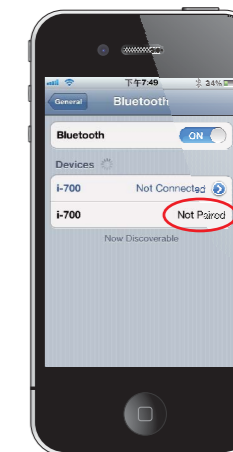

Connecting to or Disconnecting from an iPhone

- Menu** Open the menu.

- Bluetooth** Select the **Bluetooth** menu.

- Device list** The display shows all devices that have been paired before.

- iPhone** Choose the desired device.

- Connect** Or **Disconnect** Click **Settings**, **General**, **Bluetooth**, then the iPhone searches for available devices.



Once the iPhone finds **i-700**, click **Not Connected** /**Connected** to connect/disconnect it with the base. When the iPhone is successfully connected /disconnected, **Connected**/**Not Connected** appears.

Removing a Paired iPhone

- Menu** Open the menu.


- Bluetooth** Select the **Bluetooth** menu.

- Device list** The display shows all devices that have been paired before.

- iPhone** Choose the device that you want to remove.

- Remove** The display shows **Remove ? iPhone**.

- Select** A confirm tone occurs.

To delete the DECT Bluetooth information in your iPhone, click  on the right side of **i-700**, then click **Forget this Device**. **Forget Device** appears. Click it to delete.



Menu Structure

